



ADJUSTABLE DIGITOPS® with Epithelium 26®



BACKGROUND:

To relieve the pain from corns, pressure on the skin must be decreased. And this is the effect of Epithelium 26®, a weight distributor.

INDICATIONS:

- Protection of soft corns and dorsal corns after treatment

PROPERTIES:

- This toe protector is particularly recommended for **a combination of: soft corn and dorsal corn.**
- It is not stitched throughout the length of the device: At the tip of the protector there are two wings allowing for easy placement on the toe, and they can be cut to adapt the length of the toe protector to the toe.
- Epithelium 26®, present on the entire protector, is inserted between the two layers of fabric/material at the tip of the protector only for better wear resistance.



RECOMMENDATIONS FOR USE:

- Can be machine-washed at 40°C.
- Average duration of use: roughly 1 to 2 months.



References	Size	Width	Unit
0352	Small	23 mm / 0.92"	Pack of 5
0353	Medium	26 mm / 1.04"	Pack of 5

Consumer Products (with Sales Packaging)

0350	Small	23 mm / 0.92"	Pack of 2
0351	Medium	26 mm / 1.04"	Pack of 2

Skin contact compatibility:

All of our silicon gels are materials that comply with ISO standard 10993-10. They are all compatible with skin contact. No irritation, allergenic effect or cytotoxicity is observed. However, we recommend cleaning the material if applying on damaged skin. Using Epithelium in direct contact with a wound is counter-indicated. However, it can be applied by carefully inserting gauze between the wound and the Epithelium.