



ANTI-BLISTER PROTECTION with Epithelium 27™



BACKGROUND :

Over the long term, pressure and repeated rubbing on the epidermis causes separation of the dermis from the epidermis. The space created is then filled with a natural secretion: lymph. Additional pressure on the blister will cause it to break. The injured skin can then become infected, which slows the scarring process. Epithelium 27™ has continuous self-adhesive and viscoelastic properties to reduce the effects of rubbing.

INDICATIONS :

- Prevents blisters on areas subjected to rubbing
- Relieves blisters that have already formed
- Prevents or relieves irritation and burning pain caused by rubbing from clothing.

PROPERTIES :

- The bandage's self-adhesive properties mean no rubbing on the surface of the skin. This property works continuously: no adhesives or glues are used since the gel itself is adhesive.
- The bandage can be painlessly removed even in hairy areas.
- Its thin design makes it comfortable to wear with your shoes.



RECOMMENDATIONS FOR USE:

- From the sheet, cut a piece of bandage according to the size of the area you want to protect.
- We recommend applying an adhesive to keep it snugly in place.
- Remove the bandage at night so that your skin can breathe.
- The bandage can be washed by hand, so it can be **reused**.
- Duration of use: a piece of bandage can be used approximately 5 times.



References	Size	
PA271	10X10 cm / 4"x4"	10 X 10 cm sheet + 10 X 10 cm sheet of Omnifix