

# LOAD DISTRIBUTOR PATCHES

with Thuya tincture and Epithelium™



**BACKGROUND:**

**Warts are benign epidermal tumors that occur frequently (5 to 10% of the population) and are associated with a virus. A decrease in immunity as well as activities in humid environments (swimming pool, communal showers, etc.) favors the development of warts. Normally, after 2 to 3 years, your body develops resistance to fight off the virus, and the warts disappear spontaneously. However, due to the ring of horn that surrounds them, plantar warts can be painful to pressure and affect walking.**

**INDICATIONS:**


- Relieves pain for walking.
- Helps make warts disappear.
- The best results are obtained on the most recent warts.

**PROPERTIES:**

- These Epithelium™ bandages distribute pressure exerted on the wart and on the ring of hyperkeratosis that surrounds it.
- The occlusivity of the silicon gel helps to make warts disappear.
- This treatment is continuous and with no constraints.

**RECOMMENDATIONS FOR USE:**

- From the sheet, cut a section of bandage to the size of the wart and place it on the wart. We recommend applying an adhesive to keep it snugly in place.
- The bandage can be washed by hand, so it can be reused.
- Remove the bandage at night so that your skin can breathe.
- Average duration of use: 2 to 3 days. One bandage sheet can be used to treat a wart for roughly one month, depending on its size.

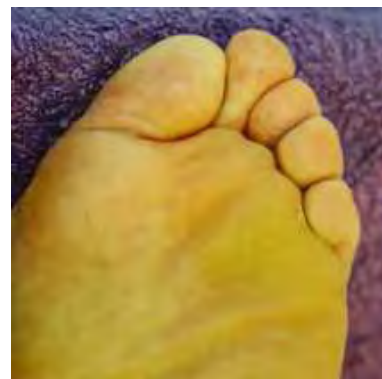

4 months later



| References | Size                               | Unit                                |
|------------|------------------------------------|-------------------------------------|
| 0724       | 1.5 mm (0.06")– single fabric side | 20 X 20 cm / 8"x 8" sheet to be cut |

Consumer Products (with sales Packaging)

|      |                        |   |
|------|------------------------|---|
| 0723 | 10 cm x 10 cm – 4"x 4" | One 10 X 10 cm (4"x4") sheet (4"x4")<br>+ one 10 X10 cm (4"x4") sheet of adhesive |
|------|------------------------|---|