



FOREFOOT LEATHER INSOLES with Epithelium 26®



BACKGROUND:

What is the ball of the foot?

The foot has a natural weight distributor called the “plantar ball”. This sub-cutaneous tissue is 1 cm thick and is made up of many fatty vesicles that deform under pressure. Unfortunately, with age there is often an irreversible change in the plantar ball which causes intense pain at the front of the foot, as well as the formation of calluses.

INDICATIONS:

- Pressure on the ball of the foot
- Pain on the ball of the foot, calluses, etc.
- Rheumatism, algodystrophy, Morton disease
- Prolonged standing, wearing high heels
- Intense activity or sport involving the forefoot

PROPERTIES:

- These leather forefoot inner soles include an Epithelium 26® module to distribute pressure in the plantar area.
- The selected lambskin leather is very comfortable and wear-resistant. It has been treated with an antifungal agent to prevent microbial growth.
- Their thin design makes them perfect for open-toed shoes or sandals and heels.
- An adhesive maintains them in place in your shoes.



RECOMMENDATIONS FOR USE:

- These forefoot insoles are worn directly against the skin in your shoes.
- The printed part of the material must be in contact with the sole of the foot.
- Slide the forefoot insole to the front of the shoe and remove the adhesive seal.
- **They cannot be washed.**
- Average duration of use: several months



References	Shoe size			Unit
	EU	UK	US	
0641	34-35	2-2.5	W4-4½	One pair
0642	36-37	3.5-4.5	W5½-6½	One pair
0643	38-39	5-5.5	W7-8 / M5½-6	One pair
0644	40-41	6.5-7.5	W8½-9½ / M6½-7½	One pair
0645	42-43	8-9	W10-11 / M8-9	One pair
0646	44-45	9.5-10.5	W11½-12½ / M9½-10½	One pair

