

LEATHER INSOLES with Epithelium 26®



BACKGROUND:

Leather insoles can easily become locations for bacterial and fungal growth (mycoses) since these organisms like hot and humid environments. So it is important to select a base material that limits microbial development.

INDICATIONS :

- Pressure on the ball of the foot
- Pain on the ball of the foot, calluses, etc.
- Rhumatism, algodystrophy, Morton disease
- Prolonged standing, wearing high heels
- Intense activity or sport involving the forefoot

PROPERTIES :

- These insoles include a module of Epithelium 26® in the plantar area.
- The selected lambskin leather is very comfortable and wear-resistant. It has been treated with an antifungal agent to prevent microbial growth.
- Thin and easy to wear in your regular shoes.
- A double-sided adhesive maintains them in place in your shoes.

RECOMMENDATIONS FOR USE:

- These leather insoles are worn directly against the skin in the shoes.
- The printed part of the material must be in contact with the sole of your foot.
- **These insoles cannot be washed.**
- Duration of use: several months on average.



References	Shoe size			Unit
	EU	UK	US	
0631	34-35	2-2.5	W4-4½	One pair
0632	36-37	3.5-4.5	W5½-6½	One pair
0633	38-39	5-5.5	W7-8 / M5½-6	One pair
0634	40-41	6.5-7.5	W8½-9½ / M6½-7½	One pair
0635	42-43	8-9	W10-11 / M8-9	One pair
0636	44-45	9.5-10.5	W11½-12½ / M9½-10½	One pair