

PLANTAR CUSHIONS DOUBLE PROTECTION with Epithelium 26®



BACKGROUND:

What is Hallux Valgus? (also known as a bunion)

Hallux valgus is a deviation of the big toe outwards, and it tends to overlap with the 2nd toe. The bone at the base of the big toe is thus further from the foot and becomes crooked, which is called a bunion. This condition is seen most often in women (90% of cases), and the main contributing factors are woman's shoes which are not adapted to the foot (narrow and pointed shoes, heels, etc.). Genetics also plays a role, particularly when this pathology affects young people. Since it changes pressure points, hallux valgus can cause plantar pain. In fact, the big toe can no longer provide adequate support and the weight it used to carry is shifted to the base of the 2nd and/or 3rd toe where calluses may be formed.

INDICATIONS:

- Pressure on the ball of the foot, plantar pain, calluses, Hallux Valgus

PROPERTIES:

- The Epithelium 26® dual-protection cushion is 2.2mm thick and acts as a substitute plantar sole. The Epithelium 26® module located under the forefoot distributes pressure in painful areas (calluses, plantar pain, pathology).
- Hallux Valgus protection is only 1mm thick, giving you the advantages of Epithelium 26® to distribute weight and prevent rubbing without causing additional pressure.
- This only provides protection and does not correct hallux valgus.
- The fabric/material is soft and absorbs perspiration, making this device very comfortable to wear. These cushions can be discreetly worn in your regular shoes.

RECOMMENDATIONS FOR USE:

- Wear directly against the skin. Wear socks, tights or stockings over the device for optimal comfort.
- To determine the correct size, measure the width of your foot at the widest point.
- Can be machine-washed at 40°C
- Average duration of use: several months



Washing bag supplied

References	Size	Foot size / width	Unit
CD 2611	Small	8 – 10 cm / 3.2" - 4"	One pair
CD 2612	Medium	10 – 11.5 cm / 4" - 4.6"	One pair
CD 2613	Large	> 11.5 cm / > 4.6"	One pair