

SHEET OF EPITHELIUM 27™



Epithelium 27™ is a self-adhesive silicon gel that can be washed and re-used.

INDICATIONS :

- As a bandage to protect against rubbing
- Protection of hallux valgus or blisters after treatment

PROPERTIES :

- Can be washed 5 to 7 times without losing its properties. It contains no glue since the gel formulation itself is self-adhesive.
- Only 1 mm thick, Epithelium 27™ is very comfortable to wear in everyday shoes.



RECOMMENDATIONS FOR USE:

- From the sheet, cut a piece of bandage according to the size of the area you want to protect.
- Place it on this area.
- Apply Omnifix or an equivalent to maintain the bandage in place.



References	Thickness	Unit
0708	1mm thick (0,04"), single fabric side	20 X 20 cm/ 8"x8" - sheet to be cut

Skin contact compatibility:

All of our silicon gels are materials that comply with ISO standard 10993-10. They are all compatible with skin contact. No irritation, allergenic effect or cytotoxicity is observed. However, we recommend cleaning the material if applying on damaged skin. Using Epithelium in direct contact with a wound is counter-indicated. However, it can be applied by carefully inserting gauze between the wound and the Epithelium.