



SHEET OF EPITHELIUM 27+™

Epithelium 27+™ is a slightly self-adhesive silicon gel that is resistant to excess pressure, and can be washed and re-used.

INDICATIONS:

- As a bandage to protect against excess pressure
- To protect a callus after treatment

PROPERTIES:

- Can be washed 5 to 7 times without losing its properties. It contains no glue since the gel formulation itself is slightly adhesive.
- At 2 mm thick, it is ideal for redistributing pressure yet comfortable to wear in shoes.
- Epithelium 27+™ comes in 2 versions: Regular or single fabric side. Epithelium 27+™ single fabric side provides the bandage with greater resistance.



RECOMMENDATIONS FOR USE:

- From the sheet, cut a piece of bandage according to the size of the area you want to protect.
- Place it on this area.
- Apply Omnifix or an equivalent to maintain the bandage in place.



References	Thickness	Unit
0709	2mm (0.08") thick, regular	20x20 cm / 8"x 8" sheet to be cut
0710	2mm (0.08") thick, single fabric side	20x20 cm / 8"x 8" sheet to be cut

Skin contact compatibility:

All of our silicon gels are materials that comply with ISO standard 10993-10. They are all compatible with skin contact. No irritation, allergenic effect or cytotoxicity is observed. However, we recommend cleaning the material if applying on damaged skin. Using Epithelium in direct contact with a wound is counter-indicated. However, it can be applied by carefully inserting gauze between the wound and the Epithelium.