



## STOCKING PROTECTORS with Epithelium 26®



### BACKGROUND:

What is the ball of the foot?

**The foot has a natural weight distributor called the “plantar ball”. This sub-cutaneous tissue is 1 cm thick and is made up of many fatty vesicles that deform under pressure. Unfortunately, with age there is often an irreversible change in the plantar ball which causes intense pain at the front of the foot, as well as the formation of calluses.**

### INDICATIONS :

- Pressure on the ball of the foot
- Pain on the ball of the foot, calluses, etc.
- Rheumatism, algodystrophy, Morton disease
- Prolonged standing, wearing high heels
- Intense activity or sport involving the forefoot



### PROPERTIES :

- These sockets, specially developed for EPITACT, are supplied with a module of Epithelium 26® in the plantar area to distribute pressure.
- The cotton fabric and elastic ribbed edges provide maximum comfort.
- Reinforced socket tips prevent wear caused by nails.

### RECOMMENDATIONS FOR USE:

- Wear directly against the skin. Wear socks, tights or stockings over the device for optimal comfort.
- Can be machine-washed at 40°C.
- Average duration of use: several months.



References	Size			Unit
	EU	UK	US	
PB 2611	35-36	2.5-3.5	W4½-6	One pair
PB 2612	37-38	4.5-5	W6½-7	One pair
PB 2613	39-40	5.5-6.5	W7½-9 / M5½-7	One pair
PB 2614	41-42	7.5-8	W9½-10½ / M7½-8½	One pair
PB 2615	43	9	W11 / M9	One pair