

## TRIPLE TIP PROTECTIONS with Epithelium 26®



### BACKGROUND:

#### What is a Quintus Varus?

**Quintus Varus is to the small toe what Hallux Valgus is to the big toe: it deviates towards the inside and joins the 4<sup>th</sup> toe. The bone at the base of the 5<sup>th</sup> toe becomes crooked, painful to pressure and eventually forms a callus. This deformation may be found alone or in combination with Hallux valgus.**

### INDICATIONS:

- Pressure on the ball of the foot
- Pain on the ball of the foot, calluses, etc.
- Pain caused by rubbing on Hallux Valgus (bunion).
- Pain caused by Quintus Varus

### PROPERTIES:

- The Epithelium 26® plantar protector is 2.2mm thick and acts as a substitute plantar sole. The Epithelium 26® module located under the forefoot distributes pressure in painful areas.
- Hallux Valgus and Quintus Varus protectors are only 1mm thick, giving you the advantages of Epithelium 26® to distribute weight and prevent rubbing without causing additional pressure.
- This only provides protection and does not correct Hallux Valgus or Quintus Varus.
- The fabric/material is soft and absorbs perspiration, making this device very comfortable to wear. It also provides warmth to the tips of toes, even with poor circulation.

### RECOMMENDATIONS FOR USE:

- Wear directly against the skin. Wear socks, tights or stockings over the device for optimal comfort.
- To determine the correct size, measure the width of your foot at the widest point.
- Can be machine-washed at 40°C in the washing bag.
- Average duration of use: several months.



Supplied with washing bag

References	Size	Foot size / width	Unit
0431	Small	8 – 10 cm / 3.2” – 4”	One pair
0432	Medium	10 – 11.5 cm / 4” – 4.6”	One pair
0433	Large	> 11.5 cm / >4.6”	One pair